

Portable Knee Joint Limit Fixation Tractor

輕便膝關節限位固定牽引器

Model : TD-KB

Features:

- Turntable design, the flexion angle can be set every ten degrees
 - Breathable fabric, light and not easy to slip
 - Opening design, easy to wear
 - Treatment of knee ligament strain and tear
 - Applicable to:
 1. Treatment of knee ligament strain and tear
 2. Fixation and rehabilitation exercise after knee surgery (under the guidance of doctors)
-
- 轉盤設計, 每十度可以設定屈曲角度
 - 透氣面料, 輕便不易滑脫
 - 開口設計, 方便穿戴
 - 適用於:
 1. 治療膝關節韌帶拉傷, 撕裂
 2. 膝關節術後, 關節鏡手術後的固定和康復鍛煉 (應在專科醫生指導下使用)



Specifications

Product Model 產品型號	TD-KB
Product Function 產品功能	fixation of knee joint 膝關節固定
Product Material 產品材質	50% polyester fiber 40% polyurethane 10% polyamide 50% 聚酯纖維40% 聚氨酯10% 聚醯胺
Fitting 支條	100% aluminum 100% 鋁
Specification 規格	L leg circumference: 53-60cm L 適用腿圍: 53-60cm
Suitable For 適用於	for left and right feet 左右腳通用
Washing Instructions 洗滌說明	hand wash, hang to dry 可手洗, 懸掛晾乾
Product Weight 產品重量	430g
Package Weight 包裝重量	730g
Packing Size 包裝尺寸	360*240*200mm

Instructions

- ◆ 刻度盤調整方法按下刻度盤兩側轉子的中間按鈕, 可依照患者需求, 調整合適的角度, 放開後即可穩定
- ◆ 按鈕兩側有安全的保護裝置, 防止按鈕受碰撞而改變固定器角度
- ◆ 將腿部放入產品內, 調整刻度盤到合適位置, 粘上魔術貼即可

注意事項:

- ◆ 未成年人、無自理能力者、淺感覺減退者需在監護人幫助下使用
- ◆ 諮詢專業人士正確使用產品, 不可擅自更改結構設計或預設角度, 定期檢查矯正器效能
- ◆ 此產品佩戴應鬆緊適度, 佩戴後如病情無明顯改善, 請及時諮詢醫生或相關專業人員