

Fully automatic electric wheelchair For the elderly

全自動老人電動輪椅

Model: LR-WBK

Features:

- High strength alloy frame, bearing 100kg, safe to use
- The new lithium battery can be carried on the plane with an endurance of 30km
- The wheelchair has a net weight of 16.5kg and is light and portable
- Anti slip design of duck foot pedal
- Fold left and right for easy folding and storage
- Manual / electric, one key switching, multi gear speed adjustable, simple operation
- Suitable for people with muscle weakness and mobility difficulties
- 高強度異形合金車架, 承重100kg, 使用安全
- 新型鋰電池, 可攜帶上飛機
- 輪椅淨重16.5kg, 輕巧又便攜
- 無刷電機質量穩定, 超長續航30KM
- 鴨掌式腳踏板防滑設計, 防止行駛過程腳部移動
- 左右收折, 折疊收納輕鬆不費力
- 手動/電動, 一鍵切換, 多檔速度可調節, 操作簡單
- 適合人群: 肌無力, 行動不便的人

Specifications

Product Model 產品型號	LR-WBK
Material 材質	aluminum alloy 鋁合金
Product Function 產品功能	out walking aid, foldable 外出助行, 可折疊
Control Mode 控制方式	manual control + electric control 手動控制+電動控制
Battery 電池	removable 5.2ah lithium battery * 2 可拆卸 5.2ah 鋰電池*2
Control Mode 控制方式	panel key + joystick 面板按鍵+操縱桿
Bearing Capacity 承重	100kg
Rear tyre 後輪尺寸	16 inches
Product Size 產品尺寸	96*64*91cm
Product Folding Size 折疊尺寸	90*35*77cm
Product Weight 產品重量	16.5kg



Instructions

◆使用說明:

1. 將腳踏板向上翻起, 坐上電動輪椅車後, 再將腳踏板放下
2. 坐上電動輪椅車後, 請系好安全帶, 打開電源, 推動操縱桿, 電動輪椅車啟動行駛
3. 操縱桿可同時控制方向和速度,
4. 若要使電動輪椅車停止, 只要鬆開手, 使得操縱桿復位即可
5. 控制器的速度調節按鈕可調節電動輪椅車的速度
6. 手推電動輪椅車時, 先把控制手柄開機再把總電源開關關閉就可以輕鬆推動電動輪椅車
7. 下車前, 先關閉控制器電源, 壓下駐力剎車, 然後將腳踏板向上翻起

◆注意事項:

1. 在閱讀和完全理解說明書前, 切勿操作您的電動輪椅車
2. 建議精神異常者, 反應遲鈍者、操縱有困難者勿使用本電動輪椅車