

# Three Piece Knee Joint Fixator

## 三片式膝關節固定器

Model : TD-KA

### Features:

- High strength aluminum alloy brace, strong fixed knee joint in extension position
- Three piece design, suitable for people of different statures
- Applicable to:
  1. Fixation of tibial plateau fracture, distal femur fracture, proximal fibula fracture, or ligament injury of knee joint before and after operation
  2. It can replace plaster or splint (it should be used under the guidance of a specialist)
- 高強度鋁合金支條, 強力固定膝關節在伸直位
- 三片式設計, 適用於不同身材人群
- 適用於:
  1. 脛骨平臺骨折、股骨遠端骨折、腓骨近端骨折, 或膝關節各個韌帶損傷手術前後的固定
  2. 可以代替石膏或夾板 (應在專科醫生指導下使用)



### Specifications

Product Model 產品型號	TD-KA
Product Function 產品功能	fixation of knee joint 膝關節固定
Product Material 產品材質	50% polyester fiber 40% polyurethane 10% polyamide 50% 聚酯纖維 40% 聚氨酯 10% 聚醯胺
Fitting 支條	100% aluminum 100% 鋁
Specification 規格	L leg circumference: 43-47cm L 適用腿圍: 43-47cm
Suitable For 適用於	for left and right feet 左右腳通用
Washing Instructions 洗滌說明	hand wash, hang to dry 可手洗, 懸掛晾乾
Product Weight 產品重量	645g
Package Weight 包裝重量	1245g
Packing Size 包裝尺寸	530*230*290mm

### Instructions

- ◆ 將支具平鋪展開, 內面朝上: 調整襯墊距離至合適的寬度
- ◆ 將患腿置於中片上, 膝關節對準開孔: 合上襯墊, 粘貼固定鬆緊程度以完全限制膝關節屈伸為宜
- ◆ 纏繞粘貼外圈綁帶, 加強固定

#### 注意事項:

- ◆ 未成年人、無自理能力者、淺感覺減退者需在監護人幫助下使用
- ◆ 諮詢專業人士正確使用產品
- ◆ 矽膠過敏者禁止使用, 使用時請注意皮膚有無破損或因受壓導致血運不暢使用部位是否出現紅腫、瘙癢、發熱、疼痛
- ◆ 此產品佩戴應鬆緊適度, 佩戴後如病情無明顯改善, 請及時諮詢醫生或相關專業人員