

# Adjustable Fixed Bracket

## 可調節固定支架

**Model: TD-TW**

### Features:

- Adjustable chuck, gradual recovery angle adjustment, simple and convenient, can walk and exercise
- Multiple non-slip straps + high-viscosity Velcro, tightly integrated with the legs, loose and adjustable
- Suitable for: leg injuries, knee injuries, ligament strains, leg corrections, etc.
- 可調節卡盤, 漸進式恢復角度調節, 簡單方便, 可行走鍛煉
- 多條防滑綁帶+高粘魔術貼, 與腿部緊密結合, 寬鬆可調
- 適用於: 腿部受傷、膝蓋受傷、韌帶拉傷、腿部矯正等



### Specifications

Product Model 產品型號	TD-TW
Product Function 產品功能	rehabilitation training / fitness 康復訓練 健身
Material 材質	aluminum alloy + sponge pad 鋁合金+海綿墊
Mode 模式	可調節 adjustable
Model 型號	左M left M
Suitable For Thigh Circumference 適合大腿圍	48-70cm
Suitable Calf Circumference 適合小腿圍	37-50cm
Product Size 產品尺寸	56*22*10cm
Product Weight 產品重量	1.05kg
Package Size 包裝尺寸	59.5*15*15cm
Package Weight 包裝重量	1.34kg



### Instructions

1. 將支具平鋪展開, 根據固定綁帶的位置, 將墊片從上到下的順序, 固定在腿上
  2. 調整好位置後, 卡盤位置正對膝關節, 將支架粘貼在腿部的左右兩側正中
  3. 將所有固定綁帶收緊, 並粘扣好即可
- 卡盤角度調節:
1. "Flex" 側調整彎曲角度, 密曲角度在0-120度之間調節
  2. "Ext" 側調整伸展角度, 伸展角度0-90度之間調節
  3. 彎曲角度於伸展角度共同作用, 可以實現卡盤的固定, 範圍活動及自由活動

#### ◆ 注意事項:

1. 最好由醫護人員或者看護人幫助固定
2. 建議先固定好支具後, 再調節角度, 這樣不易牽動傷口
3. 固定綁帶時, 注意不要太緊或太鬆, 太緊影響血液循環, 太鬆護具沒有作用