

O Leg Orthosis-leg Correction Bandage

O型腿矯正器-腿部矯正繃腿帶

Model : TD-RI

Features:

- Adjust the Velcro according to the leg type
- Thigh belt, knee belt, calf belt are widened and thicker, move, fall off
- The aluminum plate supports and fits the leg, and the strength is concentrated on both sides of the leg
- Both sides of the built-in imitation leg PVC thermoplastic version, fit the leg, force dispersion, scientific correction
- Remote control automatic pressure, no restriction on blood circulation, to avoid paralysis
- 魔術貼可根據腿型調整大小反復粘粘, 不易脫落
- 三帶合一全面覆蓋大腿帶、膝帶、小腿帶
- 鋁板支撐貼合腿部兩側, 力量集中於腿部兩側
- 兩側內置仿腿型PVC熱塑版, 高度貼合腿部輪廓, 受力分散, 矯正更科學
- 遙控自動加壓舒適矯正, 不限制血液迴圈, 避免麻痺感



Specifications

Product Model 產品型號	TD-RI
Product Function 產品功能	correct the shape of the leg 腿部矯正
Product Material 產品材質	polyester fiber, spandex 聚酯纖維, 氨綸
Voltage 電壓	12V/2A
Power 主機功率	24W/50/60Hz
Power 發熱功率	10W
Power 模式	two modes 兩種模式可換
Power 力度	9 levels 9種力度可調節
Suitable for 適用於	calf 30-55cm, thigh 40-78cm
Product Size 產品尺寸	350*460mm
Product Size 主機尺寸	150*145*55mm
Product Weight 產品重量	958g
Package Weight 包裝重量	1307g
Packing Size 包裝尺寸	45.5*19.5*13cm

Instructions

Sit and stand with legs straight, paste the Velcro in turn, and install the inflation tube in the position of the bandage inflation port and the power adapter in turn

According to their own bear step by step adjustment mode and pressure strength:

- ◆ The "+ -" button is for force adjustment, with a total of 9 levels
- ◆ Pressure relief: the emergency indicator light is always on when pressed, but not when pressed again
- ◆ Mode switching: the indicator light will flash in mode one, and it will be on for a long time in mode two

雙腿伸直坐立依次粘貼魔術貼, 將充氣管依次在綁帶充氣口和電源適配器位置安裝完成

根據自身承受循序漸進調節模式和氣壓力度:

- ◆ "+ -" 按鈕為力度調節, 共9檔力度可調節
- ◆ 洩壓: 暫停工作進行泄壓, 此時指示燈常亮, 再次按鍵恢復
- ◆ 模式切換: 模式一下指示燈閃亮, 模式二下指示燈長亮