

Elbow Joint Limit Fixed Tractor

肘關節限位固定牽引器

Model : TD-JV

Features:

- Turntable design, extension and flexion angle can be set every 10°
 - Height adjustable, width adjustable
 - Silicone gasket to prevent slippage and rotation
- Applicable to:
1. Fractures in the elbow joint, postoperative fixation and rehabilitation training of the elbow joint
 2. Treatment and rehabilitation of elbow tendon tear and ligament injury
- 轉盤設計, 每10°可以設定伸展和屈曲角度
 - 高度可調, 寬窄可調
 - 矽膠墊片, 防止滑脫和轉動
- 適應症
1. 肘關節內骨折, 術後固定
 2. 肘關節肌腱撕裂及韌帶損傷的治療與康復
 3. 腦卒中後遺症肘關節的康復訓練, 糾正上肢屈肌痙攣模式



Specifications

Product Model 產品型號	TD-JV
Product Function 產品功能	elbow joint fixation 肘關節固定
Product Material 產品材質	50% polyester fiber, 40% polyurethane, 10% polyamide 50% 聚酯纖維, 40% 聚氨酯, 10% 聚醯胺
Bracket 支架	90% aluminum, 10% ABS 90% 鋁, 10% ABS
Anti-slip Coating 防滑塗層	100% silicone 100% 矽膠
Specification 規格	Adult size 成人碼
Suitable For 適用於	29cm and above 適用範圍: 29cm及以上
Product Size 產品尺寸	53*13*10cm
Product Weight 產品重量	450g
Package Weight 包裝重量	35*23*19.5cm
Packing Size 包裝尺寸	712g

Instructions

- ◆ 刻度盤調整方法按下刻度盤兩側轉子的中間按鈕, 可依照患者需求, 調整合適的角度, 放開後即可穩定。按鈕兩側有安全的保護裝置, 防止按鈕受碰撞而改變固定器角度。
- ◆ 將手臂放入產品內, 手掌抓握住手柄, 粘上魔術貼即可

注意事項:

- ◆ 未成年人、無自理能力者、淺感覺減退者需在監護人幫助下使用
- ◆ 諮詢專業人士正確使用產品
- ◆ 凝血機制障礙、血管脆性增高及精神心理有障礙者不建議使用
- ◆ 此產品佩戴應鬆緊適度, 佩戴後如病情無明顯改善, 請及時諮詢醫生或相關專業人員
- ◆ 矽膠過敏者禁止使用, 使用時請注意皮膚有無破損或因受壓導致血運不暢使用部位是否出現紅腫、瘙癢、發熱、疼痛, 如有上述情況, 請立即停用
- ◆ 不可擅自更改結構設計或預設角度, 定期檢查矯形器效能