

# Steel tube manual wheelchair [width upgrade]

## 鋼管手動輪椅車【升級加寬款】

Model: LR-WBR

### Features:

- It can bear up to 200 kg. There are 4 brakes before and after, which is convenient for nursing and safe
- 5 seconds quick folding, just for more convenient travel
- The backrest can be folded back and can be adjusted according to different needs
- Linen cushion is breathable, easy to clean and easy to take care of
- The steel pipe frame is firm and durable, and the double support frame has strong stability
- The foot can be turned up, which is more convenient for getting on and of
- Suitable for people with muscle weakness and mobility difficulties
- 承重可達200斤, 前後共4剎車, 方便護理, 安全放心
- 5秒快速折疊, 只為出行更便捷
- 靠背可向後折疊, 可根據不同需求調節
- 亞麻坐墊透氣方便清洗, 易打理
- 鋼管車架堅固耐用, 雙支持車架穩固性強
- 腳踏可上翻, 上下車更方便
- 適合人群: 肌無力, 行動不便的人

### Specifications

Product Model 產品型號	LR-WBR
Material 材質	aluminum alloy 鋁合金
Product Function 產品功能	out walking aid, foldable 外出助行, 可折疊
Control Mode 控制方式	manual control 手動控制
Front tyre 前輪尺寸	7 inches
Rear tyre 後輪尺寸	22 inches
Product Size 產品尺寸	74*104*89cm
Product Folding Size 折疊尺寸	87*26*89cm
Product Weight 產品重量	13kg



### Instructions

#### ◆ 輪椅展開和折疊方法:

1. 雙手握住把手, 將車後輪提起離開地面, 向兩側輕拉
2. 將左右車架分開 (靠墊上沿繃直), 向下輕壓坐墊兩側至輪椅展開
3. 將腳踏板放平, 系好腿帶
4. 用手提拉坐墊中間, 使輪椅車兩側大架慢慢向中間收攏
5. 按住左右扶手外側向內壓合即可

#### ◆ 注意事項:

1. 向下壓坐墊時切勿將手指放入夾縫中, 以免受傷
2. 切勿在未剎住駐立剎車時乘坐輪椅, 以免輪椅滑動造成身體傷害
3. 切勿腳踩腳踏板上輪椅, 謹防輪椅前傾造成身體傷害
4. 切勿抓握其他部位抬動輪椅, 以免造成輪椅損壞或身體傷害
5. 下坡時, 不能使用駐立剎車, 以免翻車造成人身傷害
6. 下車時請務必先剎住駐立剎車, 駐立剎車是為使用者上下輪椅或停靠時使用, 不能用來在行駛中作制動使用